



PENGUIN ORTHOTICS

Phone:905-648-9176 Email:penguinorthotics@gmail.com

Downloading and Setting Up the Scanning App

1. Open the Apple App Store.
2. Find and download the “3DSizeMe” app (may say “Qwadra”).
3. Open the app.
4. Press the three lines in the top right corner. A menu on the left of the screen will appear.
5. Press on “Practitioners”, then “New Practitioner”. Enter your clinic information, and press “Save Information”.
6. Now, customize the app with the Penguin Orthotic “skin” by pressing the three lines in the top right corner.
7. Press “Settings”. The “Settings” window will open.
8. In the “Settings” window under “USER>Bundle ID”, press on “Click to set”. Then press the “+” icon. Type in “Penguin” and press “ok”.
9. In the same “Settings” window under “SCAN>Sensor”, make sure that “Liberty 2.0” is selected.
10. In the same “Settings” window under “IMED>Default email recipients”, enter **penguin.orthotics@outlook.com**.
11. Now close the app and open it again. This will refresh the app and install the custom Penguin “skin”. You should now see our logo in the top left corner of the screen.